

PROPER USE AND CARE OF YOUR MASK



1. Wash hands or use hand sanitizer before use.



2. Avoid touching inside of mask; ensure tight seal by adjusting straps.



3. Avoid frequent removal of mask or touching of face.




4. Be careful not to touch mask while removing.



5. Use approved disinfectant wipes or a soft cloth to sanitize interior and exterior after use.



6. Store in clearly marked bag after use, reusable up to five days.

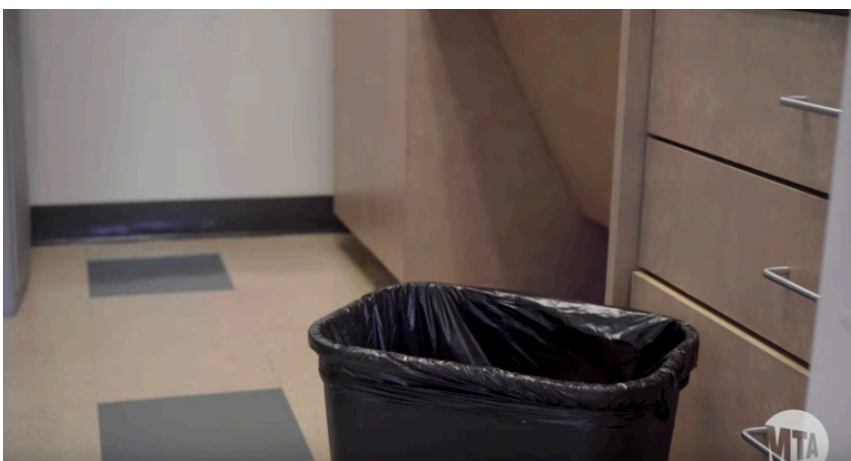

7. Discard after close contact or after 5 days of use.

For video on proper use and maintenance of N95/R95 respirators visit this link <https://youtu.be/S8BqyE2IOME> or scan below:

